

NADAL®

COVID-19 IgG/IgM Antibody Rapid Test

Your benefits at a glance:


- High sensitivity and specificity
- Easy to use
- Fast and reliable results in just 10-20 minutes
- Testing can be performed with whole blood, serum or plasma
- Test can be stored at room temperature




The NADAL® COVID-19 IgG/IgM Test is a chromatographic lateral flow immunoassay for the qualitative detection of anti-SARS-CoV-2 IgG and IgM in human whole blood, serum or plasma samples.

The test is intended as an aid in the diagnosis of primary infections and possible secondary infections.


Test Procedure:

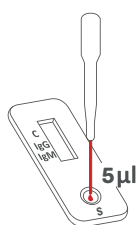

Holding the pipette vertically, draw the specimen up to the first widening (approximately 5 µL) and add it to the specimen well (S) of the test cassette.

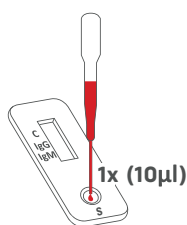

Holding the pipette vertically, add 1 drop (approximately 10 µL) of a whole blood specimen to the specimen well (S) of the test cassette.

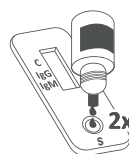

Holding the buffer bottle vertically, add 2 drops of buffer to the specimen well (S). Avoid air bubbles forming.



Wait for the red line(s) to appear. Read the test result after exactly 15 minutes. Do not interpret the result after more than 15 minutes.